

Universal Wipes

Bucket and bucket refills



Kills more than 99.99% of germs.

Clinell Universal Wipes can be used for surface cleaning and disinfection of non-invasive medical devices.

Common pathogens survive or persist on surfaces for days, weeks and months and can thereby be a continuous source of transmission if no regular preventive surface cleaning and disinfection is performed.

Clinell Universal Wipes are NOT flushable and NOT maceratable.
Please dispose of in the waste bin provided.



Bucket code: **CWBUC225AUS**
Refill code: **CWBC225RAUS**

The following surfaces & equipment should be regularly decontaminated with Clinell Universal Wipes as per local protocol/policy.

(Examples of use - this is not an exhaustive list)


- Chairs
- Desks
- Door handles
- Tap & WC fixtures
- Worktops
- Trolleys
- Incubators
- Curing lamps
- Curing lights
- Aspirators
- Couplings and tubing
- Bracket table



Conduct appropriate hand hygiene. (Risk assess PPE as per local policy)



Remove the Universal Wipes bucket lid

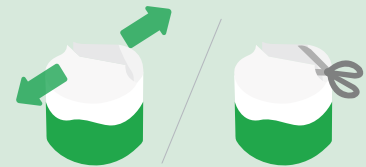

Pull open the refill pack and cut across the top with scissors



Minimising contact with the wipes, tip the refill packaging to drop the wipes into the bucket



Pull the wick through the lid slot and close the lid on to the bucket



Tick a box on the side of the bucket each time you add a refill. Replace after 5 uses

Please ensure that you follow the manufacturer's guidelines.