clinell[®] Warmer

Ensure products are kept at a consistent warm temperature, maximising patient comfort.



Convenience of Clinell

Our user-friendly warmer provides a simple top-to-toe bathing solution.

Intelligent design



Uses an accurate and reliable 'First In, First Out' system. Each shelf has a light that visually indicates when packs are ready to use and which ones should be used first. When a pack is removed for use, the light indicator is no longer present.

Even temperature



To maximise patient comfort, set the desired temperature between 37 - 45°C using the ▲▼ buttons. Quickly and easily swap between Celsius and Fahrenheit. Packs are individually positioned on each shelf, ensuring consistent heat throughout.

Compact size



Simple & user-friendly



Designed to fit on a counter top or under cupboards with a clearly visible/accessible digital display. Front-

Front-opening and transparent doors give you a clear view of products, allowing you to see how much stock is in the warmer.

Large storage area

Holds up to 24 packs.



Maximise effectiveness



To get the most out of the product, massage the pack briefly after taking it out of the warmer to ensure consistent liquid distribution among the product before opening.

Designed to be used with Clinell and Carell warmable bathing products, including wipes, wash cloths, wash gloves and shampoo caps.









First in, first out

Always use the right wipe with Clinell's intuitive 'First In, First Out' warmer design which shows when packs are ready to use and which ones should be used first.

Not ready

Indicates that packs have not yet reached the set temperature.

Ready to use

Indicates that packs have reached the required temperature and are ready for use.

Use first

Indicates that packs have been in the warmer for 60hrs and should be used first. linell' Warmer





Order information

Product	Unit of issue	Product code
Large Clinell Warmer	Single unit	CTM24

Check individual packs for heating recommendations.

