

Warmer

Ensure products are kept at a consistent warm temperature, maximising patient comfort.



Convenience of Clinell

Our user-friendly warmer provides a simple top-to-toe bathing solution.

Easy to use



Simple and easy to use. Add the packs to the warmer shelves, turn on and wait for 60 minutes. The green light will tell you when the product is ready to be used.

Even temperature



The warmer is pre-set to a pleasant 45°C (±5°C) for maximum patient comfort. Packs are individually positioned on each shelf, ensuring consistent heat throughout.

Compact size



Designed to fit on a counter top or under cupboards with a clearly visible/accessible digital display.

Simple and user-friendly



Front-opening and transparent doors give you a clear view of products, allowing you to see how much stock is in the warmer.

Large storage area



Holds up to 14 packs of Clinell Chlorhexadine and/or Clinell Bed Bathing products.

Maximise effectiveness



To get the most out of the product, massage the pack briefly after taking it out of the warmer to ensure consistent liquid distribution among the product before opening.

Designed to be used with Clinell warmable bathing products, including cloths, gloves and shampoo caps.



How to use



1




Switch on the device at the plug and ensure the green on/off switch at the back of the unit is in the ON position

2




Use the power switch on the side of the unit to turn on the device. The red light will illuminate

3

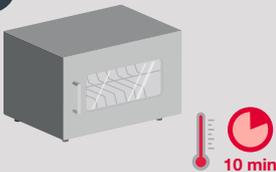


Allow the warmer to reach the pre-set temperature of 45°C (±5°C), which takes about 10 minutes

4

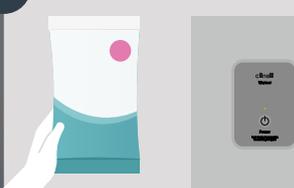

Once the temperature is reached, the red light will turn green and waterless bathing products can be placed on the shelving inside

5

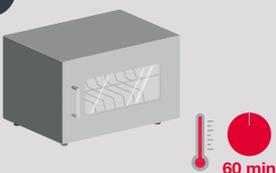

Heat the packs for 60 minutes before use



Scan here to view the warmer cleaning protocol

Order information

Product	Unit of issue	Product code
Clinell Warmer	Single unit	CWARM

Check individual packs for heating recommendations.