

2% Chlorhexidine in 70% Alcohol Skin Wipes


Pre-saturated, individually wrapped wipes designed for cleaning and disinfecting small areas of skin



Powerful cleaning formula


Wipes for cleaning and disinfecting skin after dressing and plaster removal, especially where the dressing has caused dirt to develop

Ideal for a wide range of applications and formulated with 2% chlorhexidine digluconate in 70% isopropyl alcohol, they provide thorough, reliable cleansing to help maintain skin hygiene and cleanliness.

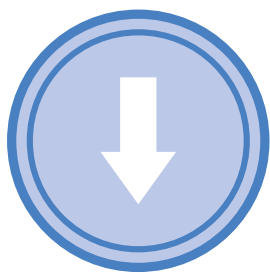


Reduces risk of infections

Cleaning with the 2% Chlorhexidine in 70% Alcohol Skin Wipes helps to reduce the bacteria that can cause skin infections

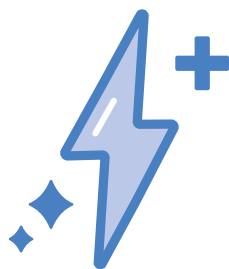

Stronger cleaning power

Effective in removing stubborn adhesive residue

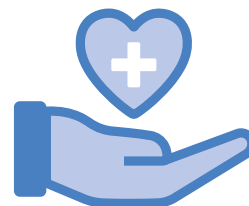

Dermatologically tested


They contain no added latex and are dermatologically tested to be used on skin

How to use

How to use Clinell 2% Chlorhexidine in 70% Alcohol Skin Wipes




Wear the appropriate PPE

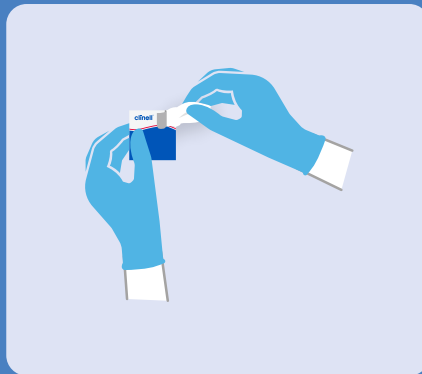


Tear open one sachet and remove the wipe



Remove any plasters or medical equipment from the area to be cleaned

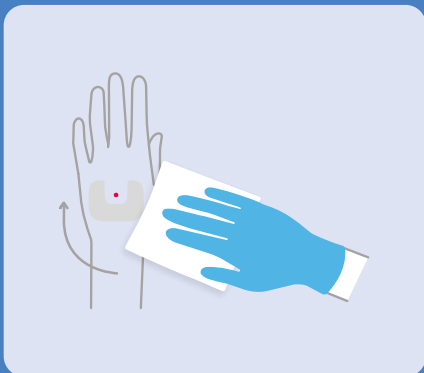


Wipe the area to be cleaned thoroughly

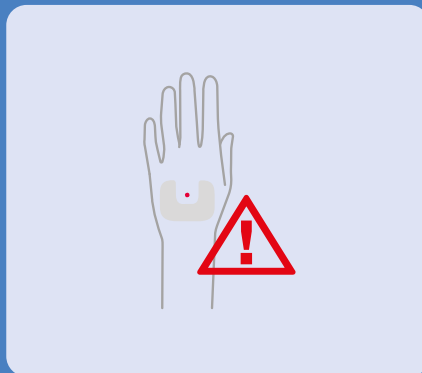

Be careful not to use the wipe on open wounds



Change the wipe if it becomes dry or soiled and discard. Discard the soiled wipe in the appropriate waste channel

	Product	Unit of issue	Product code
	2% Chlorhexidine in 70% Alcohol Skin Wipes	Box of 200	CA2CSKIN

Use biocides safely. Always read the label and product information before use.