

# Universal Wipes

## Instructions for use

**clinell**<sup>TM</sup>  
SURFACE

### Kills more than 99.99% of germs

Clinell Universal Wipes can be used for surface disinfection and cleaning of non-invasive medical devices.

Clinell Universal Wipes are **NOT** flushable and **NOT** maceratable.

Please dispose of in the waste bin provided.



CW200AUS - Universal Wipes 200

**The following surfaces & equipment should be regularly decontaminated with Clinell Universal Wipes as per local protocol/policy.**

(Examples of use - this is not an exhaustive list)

- |                |                        |                  |                                  |
|----------------|------------------------|------------------|----------------------------------|
| - Chairs       | - Tap & WC fixtures    | - Lockers        | - Blood pressure cuffs/ machines |
| - Desks        | - Worktops             | - Stethoscopes   | - Walking frames                 |
| - Door handles | - Wheelchairs & hoists | - Incubators     | - Keyboards & accessories        |
| - Mattresses   | - Trolleys             | - Walking frames |                                  |

## Instructions for use




Wash or decontaminate your hands. (Risk assess PPE according to local policy).



Remove one wipe from the pack.



Working from top to bottom, clean to dirty, wipe in an 'S' shaped pattern. Let the surface air dry to maximise contact time.

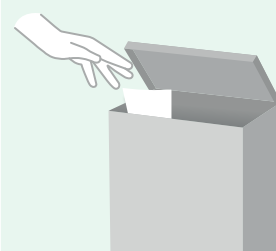

Should your wipe dry out or become soiled, replace it with a new wipe.

Please ensure that you follow the manufacturer's guidelines.