

Alcohol Wipes

Surface disinfection of non-invasive
medical devices



Rapid, high-quality alcohol wipes

Ultra-quick drying, pharmaceutical-grade 70% isopropyl alcohol wipes

Disinfection wipes, which leave no smear, for surface disinfection of non-invasive medical devices.

Proven efficacy



High-quality durable wipes containing pharmaceutical-grade 70% isopropyl alcohol. Our wipes are proven to have bactericidal and virucidal action.

Unique packaging



Wipes are dispensed one at a time, so the top of the next wipe is not dry, which commonly occurs with tub dispensers. This eliminates the infection control risk of dry, contaminated wipes.

Less plastic. Less waste



Our flow wrap packaging is environmentally friendly, as there is much less packaging to dispose of compared to standard alcohol tubs.

Superior quality



Clinell Alcohol Wipes contain 25gsm non-woven spunbond, which is wetter and thicker than competing alcohol wipes, allowing for larger area disinfection with fewer wipes, saving time and money.

For the surface disinfection of non-invasive medical devices:

- Ultrasound probes
- Pulse oximeter probes
- Doppler probes
- Stethoscopes
- ECG machines
- Blood pressure machines
- Blood pressure cuffs
- Radiology equipment
- Incubators
- Monitors

Antimicrobial efficacy



Proven to kill at least 99.999% of microorganisms

Including MRSA, *E. coli*, *Pseudomonas*, *Enterococcus*, *Candida* and *Aspergillus* according to EN 1276, EN 13727, EN 14561, EN 13624 and EN 14562.

	Microorganism example	EN test
Bacteria	<i>Acinetobacter baumannii</i>	EN 13727
	<i>Enterococcus hirae</i>	EN 13727
		EN 16615
	<i>Escherichia coli</i> K12	EN 13727
	Methicillin-resistant <i>Staphylococcus aureus</i>	EN 13727
	<i>Pseudomonas aeruginosa</i>	EN 13727
		EN 16615
	<i>Staphylococcus typhimurium</i>	EN 13727
	EN 13727	
	<i>Staphylococcus aureus</i>	EN 16615
Fungi	<i>Aspergillus brasiliensis</i>	EN 13624
Mycobacteria	<i>Mycobacterium avium</i> (2 year aged)	EN 14348
	<i>Mycobacterium terrae</i> (2 year aged)	EN 14348
Viruses	Norovirus	EN 14476
	Murine coronavirus	EN 14476
	Vaccinia virus	EN 14476
Yeast		EN 13624
	<i>Candida albicans</i>	EN 16615


Ordering information

Alcohol Wipes - Pack of 150

Transport unit view



Product code	CAW150L
Wipes per pack	150
NHSSC code (UK only)	VJT839
Expiry date	2 years
Units per carton	18
Carton size	390 x 330 x 190 mm
Cartons per layer	9
Layer height	1284 mm
Cartons per pallet	54
Classification	Class IIa Medical Device CE Mark

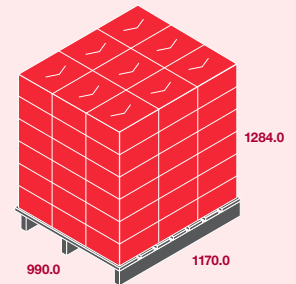

For more information, speak to your GAMA Healthcare Area Manager or visit www.clinell.com

CE 2797 Class IIa Medical Device

Conforms to the Medical Device Regulation, (EU) 2017/745.

Use disinfectants safely. Always read the label and product information before use. Always follow the medical device manufacturer's cleaning and disinfection guidelines prior to use with Clinell Alcohol Wipes.