

SPORICIDAL WIPES

Clinell Sporcidal Wipes are Peracetic acid generating wipes, activated by water. Use when *Clostridium difficile* is proven or suspected.

For use on shared patient equipment and hard surfaces.

If unsure please contact the Infection Prevention and Control Manager.




Clinell Sporcidal Wipes
(Pack of 25 - CS25AUS)

SURFACES/EQUIPMENT*

- Commodes
- Toilet facilities
- Raised toilet seats
- Door handles
- Mattresses
- Bed frames
- Bed rails
- Bedside surfaces
- Patient equipment

INSTRUCTIONS FOR USE:

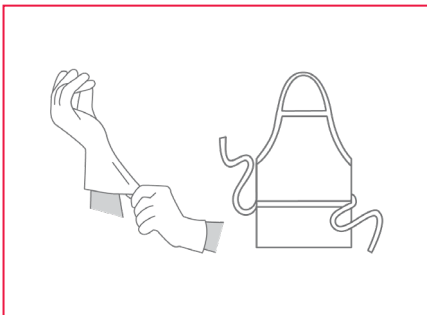


Wear recommended PPE.




Remove one wipe from packet.



Activate the wipe with water under a tap, or soak in a bucket.

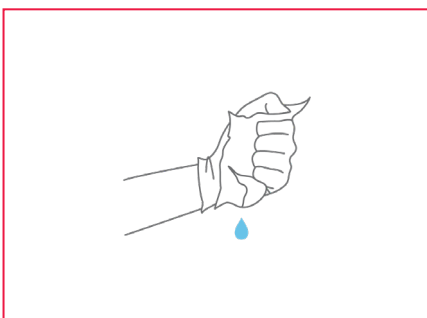

Squeeze wipe over sink or bucket to remove excess water.

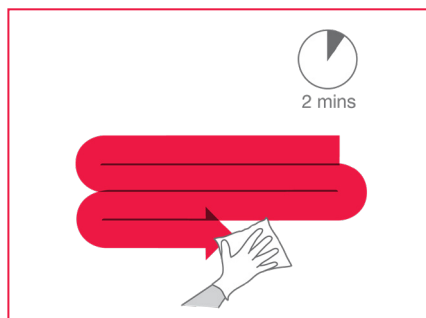

Working from top to bottom, clean to dirty, wipe in an 'S' shaped pattern, taking care not to go over the same area twice.



Dispose of wipe, gloves and apron as hazardous waste. Wash hands with soap and water.

DO NOT FLUSH DOWN TOILETS! DISCARD OPEN PACKET AFTER 35 DAYS.

* Please ensure that you follow the manufacturer's guidelines.