

Peracetic Acid Wipes

Instructions for use

clinell[™]
SURFACE

Clinell Peracetic Acid Wipes are activated by water.
Use when *Clostridioides difficile* is proven or suspected.

For use on shared patient equipment and hard surfaces. Please ensure that you follow the manufacturer's guidelines. Do not flush.

If unsure please contact the Infection Prevention and Control Manager.



(Pack of 25 - CS25AUS)

Medical surfaces and equipment

- Commodes
- Toilet facilities
- Raised toilet seats
- Door handles
- Mattresses
- Bed frames
- Bed rails
- Bedside surfaces
- Patient equipment

Instructions for use



Wash or decontaminate your hands and put on PPE (risk assessment to ensure appropriate PPE is worn, as per local protocol)



Remove one wipe from the pack




To activate, wet the wipe with water under a tap or dip into a bucket



Squeeze out the wipe to remove excess water



Working from top to bottom, wipe in an 's-shaped' pattern. Let the surface air dry to maximise contact time

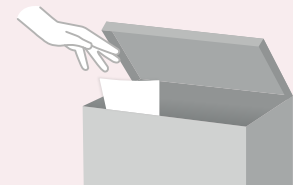

If the wipe starts to become dry or soiled, dispose of as per local protocol



Scan to see how to clean with **Clinell Sporocidal Wipes**

Expiry once opened - 35 days