

# Biodegradable Universal Wipes

## Instructions for use

**clinell**<sup>TM</sup>  
SURFACE

### Australia's first biodegradable Class IIB wipe.

To decontaminate hard surfaces, including non-critical equipment, wipe entire surface ensuring contact with all areas and allow to dry.

Effective against COVID-19 in 30 seconds.

Clinell Biodegradable Universal Wipes are NOT flushable and NOT maceratable.



Product Code: CW140PFAUS

### The following surfaces & equipment should be regularly decontaminated with Clinell Universal Wipes as per local protocol/policy.

(Examples of use - this is not an exhaustive list)

- Chairs
- Desks
- Door handles
- Mattresses
- Tap & WC fixtures
- Worktops
- Wheelchairs & hoists
- Trolleys
- Lockers
- Stethoscopes
- Incubators
- Walking frames
- Blood pressure cuffs/machines
- Keyboards & accessories



Wash or decontaminate your hands. (Risk assess PPE according to local policy)



Remove one wipe from the pack



Working from top to bottom, clean to dirty, wipe in an 's-shaped' pattern. Let the surface air dry to maximise contact time



Should your wipe dry out or become soiled, replace it with a new wipe

Please ensure that you follow the manufacturer's guidelines. The plastic lid on all flow wraps is recyclable.