

Bed Bath Cloths

Rinse-free, cleansing and moisturising cloths for a full body bathing solution.



Clinell Bed Bath Cloths

In 88 US hospitals 62.2% of bath basins were contaminated with commonly encountered hospital-acquired pathogens¹.

Safe solution

Clinell Bed Bath Cloths remove the risk of microorganism transmission associated with washbowl contamination. This reduces the associated risk of lifting and carrying heavy bowls of water and the risk of spills and potential falls¹⁻³. Improved patient cleanliness reduces the number of microorganisms present to transfer to healthcare workers, visitors and the environment.

Quick and easy to use

Cloths replace the need for traditional cumbersome patient cleansing methods which include preparing bowls, washcloths, soap, lotions and water. They require no towel drying which decreases waste, increases staff compliance and saves money.

A warm bedbath in seconds

Clinell Bed Bath Cloths can be used at room temperature or placed in a Clinell Warmer for a warm bed bath in seconds.

Full body wash

Soft cloths designed for use on each area of the body: face, neck, chest, arms, perineum, legs, back and buttocks.

A simple, rinse-free bathing solution

Designed for cleaning and moisturising the body in one easy step.

Promotes healthy soft skin

Clinell Bed Bath Cloths contain Aloe vera and vitamin E to moisturise and help hydrate skin.

Dermatologically tested

Skin neutral pH. Alcohol, lanolin and paraben-free.

Microorganism transmission associated with washbowl contamination



Washbowls are heavy and they can create spills which can lead to slips and accidents.



When a cloth is dipped back into the basin, organic matter and bacteria is introduced.



Microorganisms can be retained within the basin, thriving in wet and warm conditions.



When the basin is refilled for the next patient, the microorganisms are present within the water.


Directions for use



Bed Bath Cloths

Use one cloth per area of the body. Can be used unwarmed or heated. To heat pack, store in a warmer before use or microwave for 15 seconds*.

Use one cloth for each of the areas:

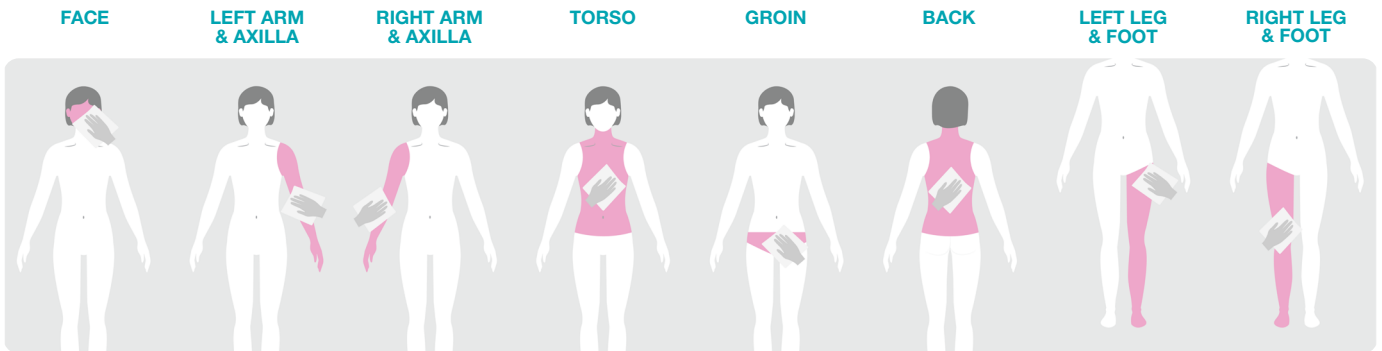

Product	Product code
 Bed Bath Cloths 8	CBB8

References:

1. Marchaim, D. et al. Hospital bath basins are frequently contaminated with multidrug-resistant human pathogens. *Am J Infect Control*. 2012 Aug;40(6):562-4. doi: 10.1016/j.ajic.2011.07.014. Epub 2011 Dec 16.
2. Johnson, D. Lineweaver, Maze, L. Patients' Bath Basins as Potential Sources of Infection: A Multicenter Sampling Study. *American Association of Critical Care nurses*. 2012.
3. Ford S. Clover, B. Antibiotic resistant bacteria risk from hospital sinks. The Department of Health. 2010.

*Microwave times may vary depending on power. Test temperature after heating and before use.
Always read the label and product information before use.