

Spill Wipes

Soaks up to 350ml of spills

clinellTM
SURFACE

Clinell Spill Wipes are specifically developed to deal with bodily fluid spills quickly and efficiently.

Soaks up spills, up to 350ml, safely in seconds.
For use on blood spills, bodily fluid spills and urine.

For more information, please contact the Infection Prevention and Control Team.



Product Code: **CSW1AUS**

Instructions for use

Wash or decontaminate your hands. (Risk assess PPE according to local policy)	Tear the pack open and remove the wipes	A is the active side and should be placed on the spill	Place wipe on the spill, with patterned B side facing up. Leave to absorb for 30 secs	Push down on the plastic backed side (B) and move around until the spill is fully absorbed
Place the soiled wipe back into the pack. Remove a disinfectant wipe from the sachet	Clean the spill in an 'S' shaped motion, from clean to dirty	Place the soiled wipe and empty sachet back into the pack	If required repeat steps 7-8 with the second wipe and re-seal	Dispose of in the appropriate waste stream, remove PPE and wash hands

DISPOSE OF IN CLINICAL WASTE! DO NOT FLUSH OR MACERATE!

Download the IFU video



Use biocides safely. Always read the label and product information before use.