

Spill Wipes

Soaks up to 350ml of spills

clinell[™]
SURFACE

Clinell Spill Wipes are specifically developed to deal with bodily fluid spills quickly and efficiently.


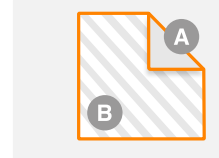
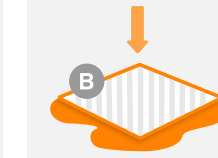
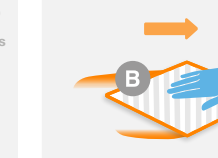

Soaks up spills, up to 350ml, safely in seconds.
For use on blood spills, bodily fluid spills and urine.

For more information, please contact the Infection Prevention and Control Team.



Product Code: **CSW1AUS**

Instructions for use

 <p>Wash or decontaminate hands and put on PPE — risk assess to ensure appropriate PPE is worn, as per local protocol.</p>	 <p>Tear the pack open and remove the wipes.</p>	 <p>A is the active side and should be placed on the spill.</p>	 <p>Place wipe on the spill, with patterned B side facing up. Leave to absorb for 30 secs.</p>	 <p>Push down on the plastic backed side (B) and move around until the spill is fully absorbed.</p>
 <p>Place the soiled wipe back into the pack. Remove a disinfectant wipe from the sachet.</p>	 <p>Clean the spill in an 'S' shaped motion, from clean to dirty.</p>	 <p>Place the soiled wipe and empty sachet back into the pack.</p>	 <p>If required repeat steps 7-8 with the second wipe and re-seal.</p>	 <p>Dispose of in the appropriate waste stream, remove PPE and wash hands.</p>

DISPOSE OF IN CLINICAL WASTE! DO NOT FLUSH OR MACERATE!

Use biocides safely. Always read the label and product information before use.

GAMA Healthcare Pty Ltd.,
Suite 1, 33-37 Duerdin St, Notting Hill VIC 3168, Australia.
+61 (03) 9769 6600 | info@gamahealthcare.com.au | www.gamahealthcare.com.au

GHA240198 **gama** healthcare