

Universal Wipes

Instructions for use



Kills more than 99.99% of germs.

Clinell Universal Wipes can be used for surface disinfection and cleaning of non-invasive medical devices.

Clinell Universal Wipes are NOT flushable and NOT maceratable. Please dispose of in the waste bin provided.



Product Code: **CW200AUS**

The following surfaces & equipment should be regularly decontaminated with Clinell Universal Wipes as per local protocol/policy.

(Examples of use - this is not an exhaustive list)

- Chairs
- Desks
- Door handles
- Mattresses
- Tap & WC fixtures
- Worktops
- Wheelchairs & hoists
- Trolleys
- Lockers
- Stethoscopes
- Incubators
- Walking frames
- Blood pressure cuffs/machines
- Keyboards & accessories

Instructions for use




Wash or decontaminate your hands. (Risk assess PPE according to local policy)



Remove one wipe from the pack



Working from top to bottom, clean to dirty, wipe in an 'S' shaped pattern. Let the surface air dry to maximise contact time

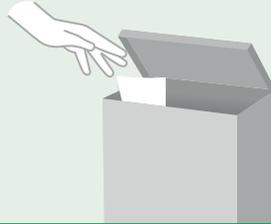

Should your wipe dry out or become soiled, replace it with a new wipe

Please ensure that you follow the manufacturer's guidelines.