

clinell®

Detergent Range

Stronger cleaning
power, ultra-low smear



Detergent Wipes

Cutting edge technology combined with ease of use to create an incredibly effective and environmentally friendly formula.

Patented formula

Safe and biodegradable formulation for an ultra-low smear clean.

Optimum cleaning

Even stronger cleaning power, developed specifically for use in healthcare environments.

Convenient and effective

Disposable wipes that are ideal for general cleaning and damp-dusting.

Multi-surface cleaning

Can be used on both surfaces and equipment.

Range of uses

(Examples of use - this is not an exhaustive list).

Surfaces

- Mattresses
- Bed frames
- Table tops
- Desks

Equipment

- Chairs
- Ledges
- Examination couches
- Telephones
- Keyboards
- Accessories
- Wheelchairs
- Trolleys
- Walking frames

Optimum cleaning

Multi-surface detergent wipes for general cleaning and damp dusting that provide a superior level of cleaning.

Clinell detergent wipes are unlike any other. Our unique formula is incredibly effective at providing optimum multi-surface cleaning and damp dusting within a healthcare setting. Many other detergent products are oil and phosphate based, which leaves smears and residue.

Our patented formula use compounds that lower the surface tension of a liquid and act as a detergent. The sufficient molecules are designed for ultra-low smearing and excellent cleaning power.

Containing no alcohol or disinfectant, Clinell Detergent Wipes are extremely safe to use and are also environmentally friendly.

How to use Clinell Detergent Wipes



Wash or decontaminate your hands and put on PPE (risk assessment to ensure appropriate PPE is worn, as per local protocol).



Remove one wipe from the pack.



Working from top to bottom, clean to dirty, wipe in an 'S' shaped pattern.



Should your wipe dry out or become soiled, replace it with a new wipe. Let the surface air dry.

Detergent clip packs

Working with the 'Design Bugs Out' initiative, our Detergent Wipes have an integrated clip, keeping wipes neatly accessible, off the floor and nearby for whenever they are needed. The integrated clip is reinforced and angled for a firm grip and securely fits on to BP machine stands, porters chairs, bed frames, monitor stands, cabinet doors, trolleys etc.



Order information

	Product	Units of issue	Order Code	NHSSC
Wipes	Detergent Wipes	Pack of 215	CDW215	VJT129
	Detergent Wipes Clip Pack	Pack of 60	CDCP60	VJT488
Tubs & Buckets	Detergent Wipes Tub	Tub of 110	CDT110	VJT225
	Detergent Wipes Refill	Refill of 110	CDT110R	VJT226
	Detergent Wipes Bucket	Bucket of 260	CDB260	VJT491
	Detergent Wipes Bucket Refill	Refill of 260	CDB260R	VJT483
Accessories	White Wall Mounted Dispenser	(for CDW215)	CDWDW	-
	Wire Wall Mounted Dispenser	(for CDT110)	CDT110D	-

For more information, speak to your GAMA Healthcare Area Manager or visit www.gamahealthcare.com.

Always follow medical equipment manufacturer's cleaning procedures and guidelines.

JBN230197

GAMA Healthcare Ltd.,

The Maylands Building, Maylands Avenue, Hemel Hempstead, Hertfordshire HP2 7TG, UK.

T: +44 (0)20 7993 0030 E: info@gamahealthcare.com www.gamahealthcare.com

