

Spill Wipes

Training and online resources



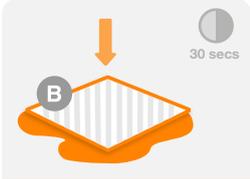
Please click on any of the thumbnails below to download or watch the provided content.

Information poster

Product brochure

How to use Clinell Spill Wipes

Instructions for use

 <p>Wash or decontaminate hands and put on PPE — risk assess to ensure appropriate PPE is worn, as per local protocol.</p>	 <p>Tear the pack open and remove the wipes.</p>	 <p>A is the active side and should be placed on the spill.</p>	 <p>Place wipe on the spill, with patterned B side facing up. Leave to absorb for 30 secs.</p>	 <p>Push down on the plastic backed side (B) and move around until the spill is fully absorbed.</p>
 <p>Place the soiled wipe back into the pack. Remove a disinfectant wipe from the sachet.</p>	 <p>Clean the spill in an 'S' shaped motion, from clean to dirty.</p>	 <p>Place the soiled wipe and empty sachet back into the pack.</p>	 <p>If required repeat steps 7-8 with the second wipe and re-seal.</p>	 <p>Dispose of in the appropriate waste stream, remove PPE and wash hands.</p>